
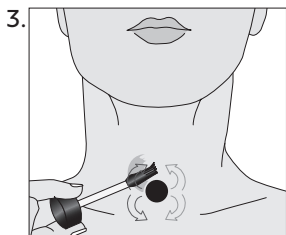
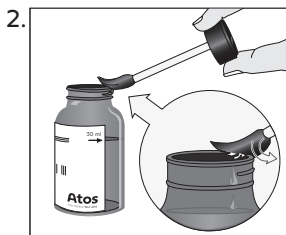
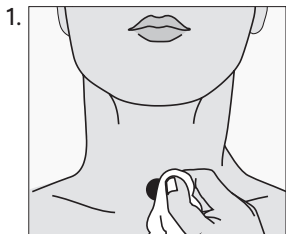
PROVOX[®]

Silicone Glue

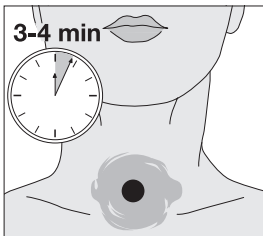
4.

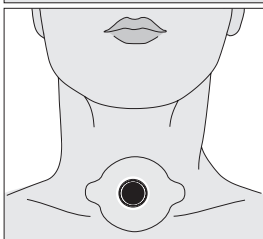5.

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Compatible Products; Kompatible Produkte; Compatibele producten; Produits compatibles; Prodotti compatibili; Productos compatibles; Produtos compatíveis; Kompatibla produkter; Kompatible produkter; Kompatible produkter; Yhteensopivat tuotteet; Samhæfar vörur; Ühilduvad tooted; Suderinami gaminiai; Kompatibilni produkty; Kompatibilis termékek; Kompatibilné produkty; Združljivi izdelki; Zgodne produkty; Produse compatibile; Kompatibilni proizvodi; Kompatibilni proizvodi; Συμβατά προϊόντα; Совместимые изделия; Uyumlu Ürünler; Համատեղելի արտադրանքներ; Produk yang Serasi; 互換性のある製品; 호환 가능한 제품; 相容產品; מוצרים תואמים; المنتجات المتوافقة

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Instructions for use

Intended use: To reinforce attachment of Provox Adhesive base plates to intact skin around the tracheostoma.

The Provox Silicone Glue is intended for single patient use.

Contraindications: Hypersensitive or damaged skin or known allergies against ingredients.

Before first use: Push down and unscrew the transportation cap. Replace with enclosed brush cap. Close the brush cap securely.

Application: Carefully clean and dry the skin around the stoma. (Preparing the skin with Provox Skin Barrier can help enhancing the adhesive properties and protecting sensitive skin from irritation). Remove excessive glue from the brush and then apply a very thin coating on the skin. If needed, apply a second layer of glue. Let the glue dry for at least 3 to 4 minutes before you attach a Provox Adhesive base plate. Close the cap securely to minimize evaporation.

Removal: Wait for spontaneous detachment. If not possible, use Provox Adhesive Remover tissues, also when taking away residual glue. Read the Instructions for Use, accompanying Provox Adhesive Remover.

CAUTION

Try to avoid deep inhalation of glue solvent (hold your breath during brush-on application) and do not permit glue to enter the tracheostoma. Solvent and fluid can cause coughing and irritation.

In case you experience skin irritation or other discomfort around the tracheostoma area, stop using the product and pause for some days. If the irritation persists, consult your physician.

WARNINGS

Keep out of reach of children.

Do not use if the brush cap applicator shows any sign of damage.

In case of eye contact, immediately remove any glue in the eye with clean absorbent cloth. Then, hold eyelids open and flush with water for at least 15 minutes. Search medical attention if irritation develops or persist or if visual changes occur.

Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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© Coloplast A/S. All rights reserved.
Atos and the Atos Medical logo are trademarks of Coloplast A/S.
Atos Medical AB