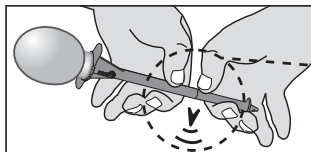
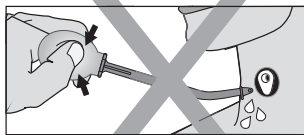
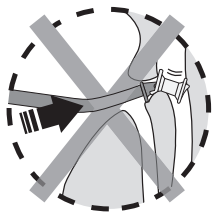
PROVOX[®] Flush


Max 25°



Max use 12 months


1	2	3	4	5	6	7
8	9	10	11	12	13	14
15	16	17	18	19	20	21
22	23	24	25	26	27	28
29	30					

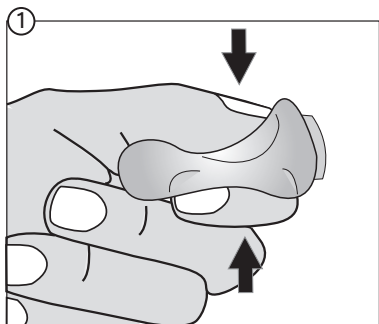
x 12


+

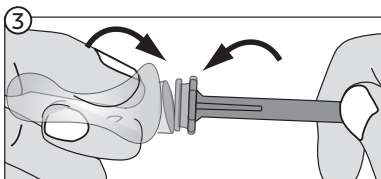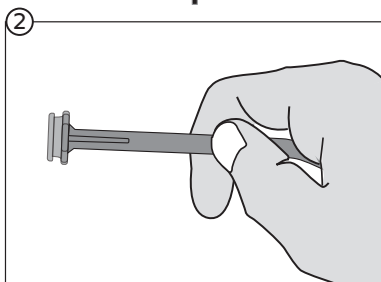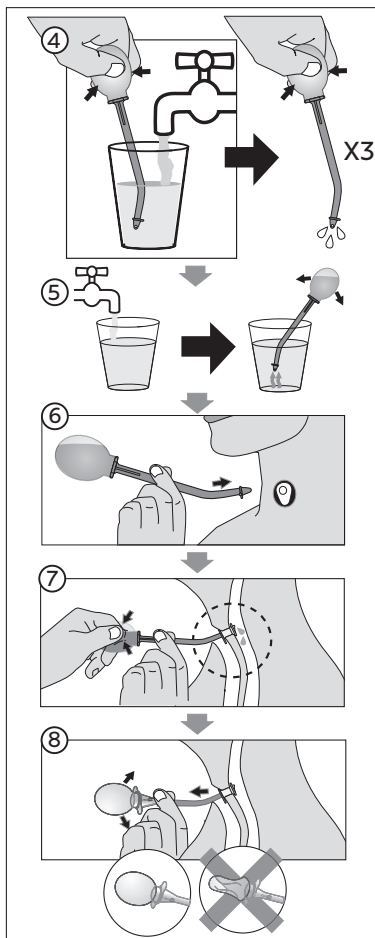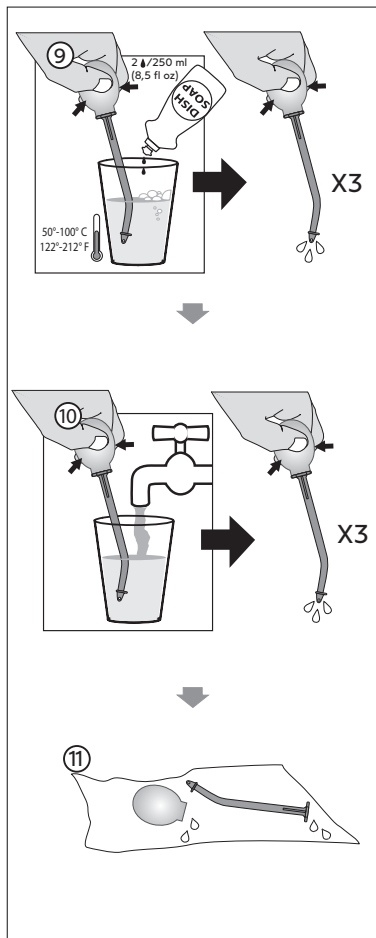12

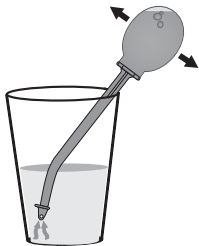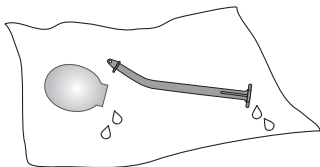
70 % Ethanol  10 min

or

70 % Isopropylalcohol  10 min

or

3 % Hydrogenperoxide  60 min

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and trademarks

Provox® is a registered trademark owned by Atos Medical AB, Sweden. For information about protective rights (e.g. patents), please refer to our web page www.atosmedical.com/patents.

Compatible Products; Kompatible Produkte; Compatibele producten; Produits compatibles ; Prodotti compatibili; Productos compatibles; Produtos compatíveis; Kompatibla produkter; Kompatible produkter; Kompatible produkter; Yhteensopivat tuotteet; Samhæfar vörur; Ühilduvad tooted; Suderinami gaminiai; Kompatibilní produkty; Kompatibilis termékek; Kompatibilné produkty; Združljivi izdelki; Zgodne produkty; Produse compatibile; Kompatibilni proizvodi; Kompatibilni proizvodi; Συμβατά προϊόντα; Совместимые изделия; Uyumlu Ürünler; Համատեղելի արտադրանքներ; Produk yang Serasi; 互換性のある製品; 호환 가능한 제품; 相容產品; מוצרים תואמים למוצרי אטוס מדיקל

Provox2 Voice Prosthesis
Provox Vega/Vega XtraSeal
Provox NID
Provox ActiValve

Contents

EN - ENGLISH.....	9
DE - DEUTSCH.....	11
NL - NEDERLANDS.....	13
FR - FRANÇAIS.....	15
IT - ITALIANO.....	17
ES - ESPAÑOL.....	19
PT - PORTUGUÊS.....	21
SV - SVENSKA.....	23
DA - DANSK.....	25
NO - NORSK.....	27
FI - SUOMI.....	29
IS - ÍSLENSKA.....	31
ET - EESTI.....	33
LT - LIETUVIŲ KALBA.....	35
CS - ČESKY.....	37
HU - MAGYAR.....	39
SK - SLOVENČINA.....	41
SL - SLOVENŠČINA.....	43
PL - POLSKI.....	45
RO - ROMÂNĂ.....	47
HR - HRVATSKI.....	49
SR - SRPSKI.....	51
EL - ΕΛΛΗΝΙΚΑ.....	53
TR - TÜRKÇE.....	55
HY - ՀԱՅԵՐԴԵՆ.....	57
RU - РУССКИЙ.....	59
MS - BAHASA MELAYU.....	61
JA - 日本語.....	63
KO - 한국어.....	65
ZHTW - 繁體中文.....	67
HE - עברית.....	70
AR - العربية.....	72

Intended use

The Provox Flush is intended to be used to flush drinking water or air through the inner lumen of a Provox voice prosthesis for cleaning purposes. The Provox Flush is intended for both home and clinical use by patient or clinician.

WARNINGS

- The flush is only recommended for patients with sufficient manual dexterity, acceptable vision and satisfactory cognitive ability.
- Do not put pressure on the voice prosthesis, the voice prosthesis may be dislodged into esophagus.
- Do not squeeze the bladder before the flush is placed into the voice prosthesis. Water will enter the trachea leading to coughing. Let the bladder be refilled with air before removing the flush from the voice prosthesis (fig 8).

Cleaning and disinfection

Clean the Provox Flush after each use according to fig. 9-11.

Do not use any water other than drinking water to clean and rinse the device.

Disinfect the Provox Flush at least once a day (fig. 12) with one of the following methods:

- Ethanol 70% for 10 minutes
- Isopropylalcohol 70% for 10 minutes
- Hydrogenperoxide 3% for 60 minutes

Caution: Do not clean or disinfect by any other method as this might cause product damage and injury.

If the Provox Flush looks dirty or has air dried in an area with a risk of contamination the device should be both cleaned and disinfected before use. A risk of contamination could exist if the device has been dropped on the floor or if it has been in contact with a pet, someone with a respiratory infection, or any other gross contamination.

During hospitalization it is important to both clean and disinfect the Provox Flush after use but also **before** use since there is an increased risk of infection and contamination. In a hospital it is better to use sterile water for cleaning and rinsing, rather than drinking water.

Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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© Atos Medical AB, Sweden.